
Quickstart Guide: Wireless Receiver


AbleNet[®]
www.ablenetinc.com

1 Insert Batteries and Turn On

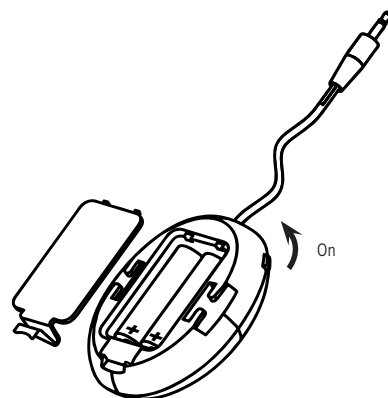
Insert two AA batteries into Receiver as shown on housing.
Slide the power switch to the "On" position. The Receiver's light(s) will activate briefly.
When batteries are low, the device's light will flash three (3) times.

2 Link Receiver & Transmitter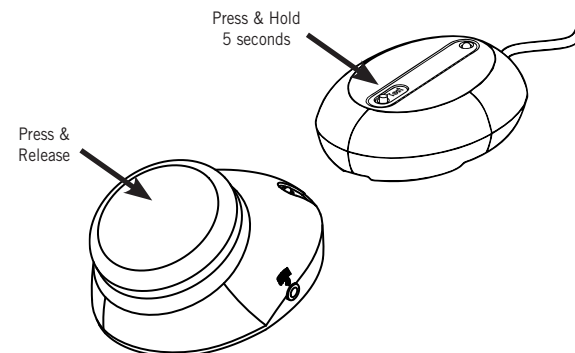

1. With wireless Transmitter and Receiver turned on, press and hold the Receiver's "Test" or "Mode" button until the light turns green (about 5 seconds).
2. Activate any Transmitter. *The Receiver's light will flash red 1 time when a link has been established.*
**Link additional Transmitters by activating them now (optional).*
3. Press and release the Receiver's "Test" or "Mode" button to exit linking mode.

3 Connect Receiver & Activate

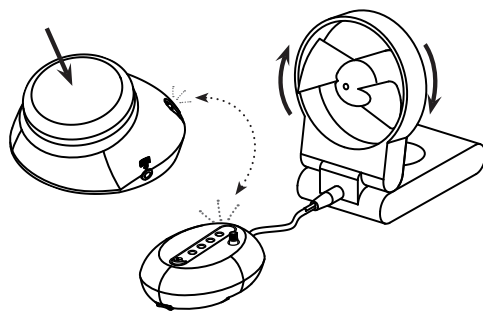Plug the Receiver into any switch-accessible device or computer switch interface.

Activate the Transmitter.

The Transmitter and Receiver will light red and the attached device will activate.

NOTE: To test the physical connection with a switch-adapted device, press and hold the "Test" or "Mode" button for 2 seconds.

Tips

Your Receiver has slots on the bottom of the device for mounting with Velcro[®] straps. For additional mounting options, visit www.ablenetinc.com.

You can link up to 8 Transmitters to a Receiver using the linkable method explained in Step 2. When a Receiver's light is green, any activated Transmitter will be linked. One Transmitter can also be linked to multiple Receivers at the same time.

To unlink devices, press and hold the Receiver's "Test" or "Mode" button for more than 10 seconds.

To clean, wipe surfaces with a slightly damp cloth.

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Warranty & Contact Information

AbleNet manufactured products include a two-year limited warranty. This warranty is against defects in materials and manufacturing for two-years from the date of purchase. Warranty in full text, along with return/repair information is available at www.ablenetinc.com.

AbleNet, Inc.
2625 Patton Road
Roseville, MN 55113
United States of America
(800) 322-0956
(651) 294-2200
www.ablenetinc.com

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For private households:

Information on Disposal of Users of WEEE

This symbol  on the product and/or accompanying documents means that used electrical and electronic equipment (WEEE) should not be mixed with general household waste. For proper treatment, recovery and recycling, please take this product to designated collection points where it will be accepted free of charge. Alternatively, in some countries, you may be able to return your products to your local retailer upon purchase of an equivalent new product.

Disposing of this product correctly will help save valuable resources and prevent any potential negative effects on human health and the environment, which could otherwise arise from inappropriate waste handling. Please contact your local authority for further details of your nearest designated collection point. Penalties may be applicable for incorrect disposal of this waste, in accordance with your national legislation.

For professional users in the European Union:

If you wish to discard electrical and electronic equipment (EEE), please contact your dealer or supplier for further information.

For disposal in countries outside of the European Union:

This symbol is only valid in the European Union (EU). If you wish to discard this product, please contact your local authorities or dealer and ask for the correct method of disposal.

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FCC Title 47 Part 15b

Instructions for Use Information

Note: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult AbleNet for help.

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